T-Pessar

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CE 0483

Made in Germany

M E D E S I G N
Dietramszell · Dubai · München

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	Deutsch	4
	English	11
	Français	19
	Italiano	27
	Español	35
	Türkçe	43
	Polski	51
	Русский	59
	中文	67
	يبرع	75

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The background of the entire page is a solid blue color. On the left side, there is a blurred, light blue silhouette of a woman's back and shoulders. On the right side, there is a close-up of a sunflower with bright yellow petals and a dark brown center.

T-Pessary

 Instructions for use

When is therapy with a pessary useful?

T-Pessaries are used for treating certain types of incontinence (inability to control urination or defecation), pelvic floor disorders (often a combination of cystocele, enterocele and rectocele) as well as problems caused by descensus of the vagina, the uterus, the bladder or the intestine. A cystocele appears when the very thin wall between the bladder and the vagina becomes too weak and the bladder falls down into the vagina. This makes the complete emptying of the bladder impossible and mostly presents itself together with stress incontinence. Descents like this can be caused by e.g. hereditary weakness of the connective tissue, postmenopausal hormone deficiency or pregnancy and giving birth. Treatment with pessaries - combined with pelvic floor exercise and, if necessary, oestrogen enriched creams - can permanently improve conditions and allow changing to a smaller sized pessary or even totally obviate the need for a pessary.

How does the T-Pessary work?

The cylindric shape of the T-Pessary repositions the organs and brings them into the anatomically correct position – counteracting a prolapse or descensus of the vagina, the uterus, the bladder or the intestine. Therapy with pessaries is an alternative to surgical intervention for the treatment of descensus uteri and incontinence. The T-Pessary keeps the bladder and the intestine in their anatomically correct position - particularly during physical strain such as hard work, doing sports, etc.

Which pessary is the right one for me?

medesign's T-Pessaries Type B and Type R are offered in various sizes (see table). The pessary, which is smallest in size, yet resting securely should be chosen. Selecting the correct size is a prerequisite for ideal treatment and freedom from symptoms. Your gynaecologist will determine the right size for you.

The pessary is inserted and you can check its correct position by moving, coughing and squeezing, whereupon the gynaecologist will check the fit of the pessary once again. The pessary, which is smallest in size, yet resting securely should be chosen. You should also make sure that you are able to urinate without any problems.

Your gynaecologist will assess the fit of your pessary after a few weeks or months to see whether you can switch to a different size. The various sizes allow for individual fitting. The specifically designed surface prevents pressure marks on the mucous membranes to the greatest extent. T-Pessaries are available perforated (Type B) or corrugated (Type R).

medesign pessaries are made from flexible, tissue tolerant silicone, which is especially developed for medical use. If you should, despite this, notice a reaction, stop the use of the T-Pessary and discuss the situation with your physician. The pessaries are flexible and can thus be folded and squeezed, making insertion and removal especially easy. The pessary may be used by one patient only.

How do I insert the T-Pessary?

Pessaries can easily be inserted on your own. Check the T-Pessary for intactness, cleanliness, discolouration, damage and wear marks. The pessary has to be replaced by a new one should damage or changes on the surface be visible.

You should insert the T-Pessary into the vagina in the mornings and remove it in the evenings in order for the sensitive vaginal tissue to be able to recover overnight. In certain cases the pessary may remain inserted during the night or

be replaced by a smaller pessary at night. Please follow your doctor's instructions for this.

Wash your hands prior to inserting the pessary. To ease insertion it is recommended that a lubricant, such as Ladysoft, be used. If needed, the T-Pessary may be coated lengthwise with a cream or gel containing oestrogen.

Now insert the pessary into the vagina like a tampon with the string hanging down. It may be helpful to place one leg e.g. on the edge of your bed. Some women find it easier to insert the pessary in a crouching or lying position. You could also lean against a wall with your legs slightly spread and then insert the pessary. Try out which method is best for you.

Push the T-Pessary as far into the vagina as possible or into the position shown to you by your physician. The pessary is placed correctly if you cannot feel any pressure.

How do I remove the T-Pessary?

You should also wash your hands prior to removal. Thoroughly lubricate your vaginal area with Ladysoft Lubricant. Use the attached string to pull the T-Pessary downwards until you feel a slight resistance from the pelvic floor muscles. It is best to crouch down and gently pull the string into different directions. Using the string (like with a tampon), the T-Pessary can be removed from the vagina. Do not use any other equipment for removing the T-Pessary.

What do I do if the string tears?

You can use your fingers to remove the pessary, in the exceptional case of a torn string. If you are not able to remove the pessary, let your physician remove it within the next few days.

Pessary Care

Clean the T-Pessary with lukewarm water and soap or a mild washing lotion such as Femsoft after removal. Subsequently rinse and leave to dry. Disinfection is usually not necessary. Wash your hands again after cleaning the pessary.

For added support for the cleaning and care of you intimate area, use e.g. Femsoft Intimate Wash – a specially developed washing lotion with whey and stinging nettle as a basis and a low pH-value (3.8 – 4). Subsequently wash your hands again.

**Do not use chemicals for cleaning.
Do not use household or dish detergents.
It is not necessary to boil the pessary.
Do not put it in the microwave.**

Disposal

medesign pessaries are made from certified medical silicone and can be disposed of in household waste. The silicone is not biodegradable and should therefore not be disposed of in the toilet.

Infections of the vagina and the cervix are contraindications for pessary treatment.

Please note

- Read and follow instructions for use.
- Use according to instructions from healthcare professionals.
- Uninterrupted period of application without medical supervision: 29 days maximum.
- Maximum period of application with medical supervision: 2 – 3 cycles of 29 days maximum.
- Repeated use with the same patient possible.
- Tolerance and correct placement of the pessary have to be secured through ultrasound and / or consultation.
- The product has to be checked for intactness and cleanliness of the surface prior to every application. The pessary has to be replaced by a new one should changes on the surface appear.
- The pessary has to be cleaned after every application – using lukewarm water and soap and subsequently rinsing the pessary with clear water.
- The periods of application have to be documented by a physician.
- Do not use during menstruation and never use the pessary together with a tampon or without medical consultation.
- Do not use during pregnancy or if you think you might be pregnant.
- Do not use if you have a bladder or urinary tract infection or inflammation of the vagina.
- medesign pessaries are latex-free.

Additional products

Ladysoft Lubricant: eases insertion of the pessary

Femsoft Intimate Wash: gentle cleansing of the intimate area

Femsoft Intimate Cream: nourishes and protects the intimate area

Usable until

LOT

Date of construction

Producer

Item number (MED...)

Attention

Read and follow the instructions for use

CE-mark with identification number
of the notified body

T-Pessary Type B	Size	Diameter	Length	Article No.
	Size 1	Ø 20 mm	53 mm	MED1001649
	Size 2	Ø 25 mm	56 mm	MED1001640
	Size 3	Ø 32 mm	78 mm	MED1001650
	Size 4	Ø 36 mm	78 mm	MED1001727
	Size 5	Ø 42 mm	78 mm	MED1001728
	Size 6	Ø 46 mm	81 mm	MED1001729
Fitting Set: Sizes 2, 3, 4, 5				MED1001733

T-Pessary Type R		Diameter	Length	Article No.
		Ø 20 mm	50 mm	MED1001651
	*	Ø 25 mm	55 mm	MED1001641
		Ø 32 mm	50 mm	MED1001810
		Ø 32 mm	62 mm	MED1001811
		Ø 32 mm	72 mm	MED1001812
	*	Ø 32 mm	79 mm	MED1001652
		Ø 36 mm	50 mm	MED1001813
		Ø 36 mm	62 mm	MED1001814
		Ø 36 mm	73 mm	MED1001815
	*	Ø 36 mm	80 mm	MED1001724
		Ø 42 mm	55 mm	MED1001816
	*	Ø 42 mm	68 mm	MED1001817
		Ø 42 mm	76 mm	MED1001808
		Ø 46 mm	60 mm	MED1001818
		Ø 46 mm	73 mm	MED1001819
		Ø 46 mm	82 mm	MED1001809
Fitting Set: Sizes with * included				MED1001732

Content:

- 1 pessary with instruction manual
- 1 Ladysoft Lubricant with instruction manual

If you have any questions regarding application or would like to share your experiences, please contact us.

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medesign I.C. GmbH