Ayre Spatel

Plastik Pap Plus

50 Stück / pieces

Plastik Zervix Abstrichbürsten
Plastic cervix swab brushes
Brosses à frottis cervicales en plastique
Spazzole tampone per cervice in plastica
Cepillos de plástico para cervix

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 Temperaturgrenzwerte

SRN Single Registration Number

medesign Ayre Spatula plastic Pap Plus Directions for Use

Introduction

Please read the Instructions for Use carefully before you start using this medical device, as they contain important information. Always use this medical device exactly as described in these Instructions for Use.

Follow the safety instructions.

Purpose

For vaginal/ectocervical sampling for histology/cytology.

Product description

The Ayre Spatula Plastic Pap Plus gently scrapes the mucosa to ensure a rich ectocervix sample. The polypropylene plastic is inert and allows easy transfer of the collected cells to the slide.

Safety instructions

- For use by trained personnel only.
- The product may only be used once. Dispose of after use.
- Non-sterile.
- Do not use the medical device if it is damaged.

Indications

The Ayre Spatula Plastic Pap Plus is used to collect cell samples from the ectocervix for Pap smear or liquid-assisted cytology.

Contraindications

Not applicable in patients with heavy bleeding (no safe cytology possible). Take appropriate precautions if infection or sexually transmitted diseases are suspected.

Side effects

There are no known side effects.

Application notes

- For use by trained personnel only.
- The product may only be used once. Dispose of after use.
- Non-sterile.
- Not applicable in patients with heavy bleeding (no safe cytology possible). Take appropriate precautions if infection or sexually transmitted diseases are suspected.

Instructions for taking the sample

- Carefully fix the tip of the spatula in the center of the portio vaginalis.
- Slowly complete a full 360 degree rotation with the spatula.
- Pull out the Ayre Spatula plastic Pap Plus.

Preparation of the specimen for Pap smear cytology

- Smear the spatula evenly over a clean glass slide leaving room for the endocervical sample.
- Fix immediately with cytology spray fixative or 95 % ethanol.

Preparation of liquid-assisted cytology

Consult your local laboratory for detailed instructions.

Storage

- Application temperature 15-40 °C.
- Protect from direct sunlight.

Shelf life

Expiration date of the product according to the product labeling.

Disposal/environmental protection

After completion of treatment, dispose of the Ayre Spatula according to the waste disposal plan of the medical facility. Observe your local and national laws, guidelines, standards and specifications for disposal.

Our products

Item-Nr.	Item name
REF MED1000893	Ayre plastic spatula ‚Pap Plus‘, 200 pieces
REF MED1002029	Ayre plastic spatula ‚Pap Plus‘, 50 pieces

**Note according to the requirement of
Regulation (EU) 2017/745 MDR**

In case of questions and problems in connection with the Ayre Spatel as well as in case of serious complications which can be traced back to the Ayre Spatel, please contact the manufacturer medesign I. C. GmbH, info@medesign.de or Tel.: +49 (0) 8027 907380. Please report to us immediately if a serious incident has occurred in connection with the product which directly or indirectly had, could have had or could have had one of the following consequences:

- The death of the patient, user or other person,
- the temporary or permanent serious deterioration of the health status of a patient, user or other person,
- a serious public health hazard.

In case of a serious incident, you can also contact the competent authority of your country.

Symbols

REF Item number

LOT Batch number

 Do not reuse

 Non-sterile

 Manufacturer

 Observe operating instructions

 Usable till

 **Attention**

MD Medical device

UDI Unique product identification

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medesign I.C. GmbH