Kirschkernkissen

Cherry Pit Bag

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Cherry Pit Bag

To warm and feel good

medesign's Cherry Pit Bag is a traditional household remedy used in the same way as a hot water bottle. Therefore it also goes by the name of waterless hot water bottle.

When the Cherry Pit Bag is heated up, the pits inside the bag will retain heat much longer than water in a hot water bottle. Due to its design, the Cherry Pit Bag will perfectly adapt to the body's shape allowing for perfect thermal transmission.

- Heating: In the oven at a maximum of 80 °C
- Briefly in the microwave - without grill and Hot air function (risk of fire!)
- Warning: risk of overheating
- Never heat up cherry pit pillows unattended
- Warming by adults only, not a toy
- Check for optimal heat before use

Start with the lowest setting and increase it as needed until the cushion is comfortably heated to approximately 42-58 °C. (please follow the operating instructions for your microwave). If less heat is required, the pillow can also be placed on the heater.

Cold application: The medesign cherry pit cushion is also suitable as a cold compress. To do this, simply cool the bag to the desired temperature in the refrigerator or freezer.

The medesign cherry pit cushion has another very pleasant advantage: it is particularly suitable as a massage cushion for pregnant women and for general finger and foot gymnastics, especially when it is important to restore the mobility of the limbs through a targeted massage or to activate it in children.

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medesign I.C. GmbH