Nabelbruchband

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Navel Hernia Belt for newborns

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Nicht Steril

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Handwäsche

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The umbilical hernia band is a medical device. Please read these instructions for use carefully before use. If you have any questions, please contact your doctor or the specialist retailer where you obtained this medical device.

Intended purpose

For the conservative treatment of umbilical hernias in newborns.

Indication

Umbilical hernia in newborns.

Contraindication

Unclear soft tissue swelling, sensory and circulatory disorders in the treated body part, skin disorders in the treated body part.

Precautions

- Remove the umbilical hernia band before radiological examinations.
- Avoid impairing the blood circulation when wearing.
- Do not make any changes to the product.
- Do not use on open wounds.
- Do not reuse - only use this medical device on a newborn.

Fitting instructions

Lay down flat bandage (long white fastener downwards). Put bandage with necessary tension from the back around the body. Put the small loose piece over the naval which is covered with a sterilized gauze-compress. Now put the piece with the big hook-and-loop tape across and close with the small fastener by pressing against.

Removal

To remove, open the fastener, then remove the bandage.

Material composition

54% cotton, 46 % polyamide.

Cleaning instructions

Hand wash at 30 °C.

Do not tumble dry.

Do not dry clean.

Close the fastener to avoid damaging other items of clothing. Do not use fabric softener. Air dry.

Durability

Shelf life according to product labeling.

Disposal/environmental protection

After use, the product must be disposed of in accordance with local regulations.

Mandatory information according to Regulation (EU) 2017/745 (MDR)

If you have any questions or problems in connection with the umbilical hernia band or in the event of serious complications attributable to the umbilical hernia band, please contact the manufacturer medesign I. C. GmbH, info@medesign.de or phone: +49 (0) 8027 907380. Please notify us immediately if a serious incident has occurred in connection with the product. A serious incident includes both the risk and the actual occurrence of two events: Death or serious deterioration in the state of health of a patient, user or third party. In the event of a serious incident, you can also contact the competent authority in your country.

Symbol legend

Manufacturer

Medical device

CE mark

Attention

Single Patient
multiple use

Instructions for use observe

Without latex

Date of manufacture

Batch number

Can be used until

Not sterile

Wash at max. 30 °C

Hand wash

Do not tumble dry

Do not dry clean

PRN21042 SP00482.004  702881

medesign I.C. GmbH · Dietramszeller Str. 6 · D-83623 Dietramszell
www.medesign.de · info@medesign.de