Rapssamenkissen

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Rape Seed Bag

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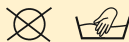
Rape Seed Bag

for gentle massage and thermo-therapy

The fine rape seeds inside the rape seed bag can be heated and – compared to conventional hot water bottles – can store and release the heat for much longer.

When the Rape Seed Bag is heated up, the pits inside the Bag will retain heat much longer than water in a hot water bottle. Due to its design, the Rape Seed Bag will perfectly adapt to the body's shape allowing for perfect thermal transmission.

Heating: In the oven at a max. temperature of 100 °C.
In the microwave, without grilling or
hot air function (danger of combustion!)

Start with the lowest level and increase gradually until the cushion has been heated up to a pleasant temperature. Please check your microwave or oven manual. Where less heat is needed, the cushion can also be laid on the radiator.

Cooling: medesign's Rape Seed Bag may also be used to provide cooling. Simply put the Bag in a freezer or refrigerator.

Furthermore, the medesign Rape Seed Bag can be used in massages during pregnancy or for general hand and foot exercises.

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medesign I.C. GmbH