Siebschalen-Pessar Siebschalen-Pessar EF (Extra Fest)

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Made in Germany

MEDESIGN

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Deutsch	4
English	14
български	24
Dansk	36
Eesti	47
Suomi	57
Français	67
Ελληνικά	78
Italiano	90
Hrvatski	101
Latviešu	111
Lietuvių	121
Nederlands	131
Polski	142
Português	153
Română	164
Svensk	175
Slovensky	185
Slovenščina	195
Español	205
Česky	216
Magyar	226
Türkçe	237
Русский	248
中文	260
قبرعلل ءغلل	279

1. Produktbeschreibung

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Sieve bowl pessary
Sieve bowl pessary
EF (extra firm)

 Instructions for use

1. Product description

- medesign pessaries are medical devices made of silicone or silicone foam, which are available in different types and sizes.
- They are inserted into the vagina and support the pelvic floor organs.
- All medesign pessaries are latex-free and do not contain phthalates.

2. Safety instructions

- The product may only be used by one patient. It can be used multiple times by one patient.
- The pessary should not be worn continuously for long periods of time without medical supervision in order to prevent serious complications and avoid the need for surgical intervention.
- No pessary therapy if the patient is pregnant and experiencing active labor, onset of labor, or premature rupture of membranes.
- No postpartum pessary therapy in cases of placental remnants, vaginal edema formation, or suture dehiscence.
- Pessary therapy with existing surgical mesh in the vagina: The physician decides whether the pessary can be inserted, whether medical follow-up must take place at shorter intervals, and whether there is a possibility that the mesh could be affected by the pessary.
- Before each use, check the pessary for integrity and cleanliness. If the surface has changed, replace the pessary with a new one.
- Do not use the pessary if you have an infection of the vagina and/or cervix.
- TSS information: Although pessaries are not known to increase the risk of TSS (toxic shock syndrome), caution should be exercised. Symptoms: If you suddenly develop a fever, chills, nausea, vomiting, muscle pain, drop in blood pressure, dizziness, or lightheadedness, remove the pessary as a precaution and seek immediate medical attention from a doctor or clinic. Mention TSS and that you have been using a pessary.

- Do not use if you have or suspect you may have an allergy to solid silicone.
- If the pessary is used together with vaginal preparations (e.g., hormone ointments, vaginal creams containing lactic acid), this may discolor or damage the surface of the pessary under certain circumstances. Water-based lubricants, which facilitate insertion of the pessary, are an exception.
- The pessary must be removed before an X-ray, MRI, or CT scan, as it can negatively affect the results of the examination.

3. Performance characteristics

- Pessaries for relieving pressure on the pelvic organs are divided into two categories: supportive and space-filling. medesign pessaries are available in different types and sizes.
- The pessaries can usually be easily inserted, removed, and cleaned by the patients themselves.
- Once the doctor has selected the right type and size of pessary for the patient, the pessary will not be noticeable.
- A pessary provides immediate relief from the symptoms of prolapse-related discomfort or stress urinary incontinence by mechanically lifting the pelvic organs and thus restoring their closing function.
- The benefits for the patient are felt very quickly, without surgery or pain.

4. Indication/Intended use

Pessaries are used to treat symptoms caused by uterine prolapse and to relieve pressure on the bladder. Uterine prolapse can be caused, for example, by congenital connective tissue weakness, hormone deficiency in postmenopausal women, or pregnancy and childbirth.

5. Target patient group

Women of childbearing age who suffer from pelvic organ prolapse and possibly resulting incontinence.

6. Clinical benefit and expected clinical outcome

- Clinical benefits: medesign pessaries reposition the pelvic floor organs into their anatomically correct position and counteract prolapse or descent of the uterus (uterine prolapse), bladder (cystocele), or rectum (rectocele).
- Expected clinical outcome: Pessaries alleviate the symptoms of uterine prolapse, stress urinary incontinence, cystocele, rectocele, and nocturia with urinary retention.
- Pessary therapy is an alternative to surgery for the aforementioned conditions and symptoms.

7. Duration of use

- Maximum continuous use without medical supervision: 29 days.
- Maximum period of use under medical supervision: 2 to 3 cycles of up to 29 days each.
- We recommend removing the pessary every evening and reinserting it in the morning.

8. Contraindications

- Do not use during menstruation and never together with a tampon, or only after consulting a doctor.
- Do not use during pregnancy/suspected pregnancy or only after consulting

a doctor.

- Do not use in cases of infection, cancer, lacerations, or eroded surgical mesh in the internal pelvic area.
- For patients who are unable to change and clean the pessary regularly (due to needing care, dementia, or other physical/mental impairments), it is the responsibility of family members to find a solution for the patient in consultation with the treating gynecologist. If necessary, a family member or caregiver should be involved in changing and cleaning the pessary. The same applies to patients who are unable to understand medical information and instructions or who cannot be monitored.

9. Side effects

In case of dysuria, pelvic pain, mucosal ulcerations, bleeding, or abnormal vaginal discharge or odor, or in case of allergic reactions to solid silicone, use must be discontinued and the treating physician consulted.

10. Cleaning recommendation

To clean the pessary, use water (drinking water quality) and a mild, ready-to-use washing lotion, such as the medesign cleaning solution for pessaries. Follow the manufacturer's instructions.

- Step 1:** Rinse the pessary for 10 seconds under running cold water (<40°C, drinking water quality).
- Step 2:** Place the pessary in a water bath with a mild washing lotion (5 ml/liter), e.g., medesign cleaning solution for pessaries, and then clean the pessary manually for approx. 5 minutes until no visible residues remain.
- Step 3:** After cleaning, rinse with cold water (<40°C, drinking water quality) for at least 10 seconds to remove any soap residue. If any dirt

remains visible, repeat the cleaning process.

Step 4: Dry the pessary with paper towels or a lint-free towel. The clean pessary can then be left to dry further either in an air-permeable container or in the open air. Wash your hands again after cleaning the pessary.

Once dry, the pessary can be reinserted immediately.

The pessary can be disinfected using a commercially available liquid disinfectant (e.g., octenisept® vaginal therapeutic agent or octenisept® wound disinfectant). We recommend disinfection after an infection of the bladder, urinary tract, or vagina (e.g., vaginal yeast infection).

Do not clean with chemical agents. Do not use household cleaners or dishwashing detergent. It is not necessary to boil the pessary. Do not place the pessary in the microwave.

If there are changes to the surface, the pessary must be replaced with a new one.

11. Size selection

Choosing the right size is essential for optimal treatment and symptom relief. medesign pessaries are available in different sizes, allowing for individual adjustment for each patient. The pessary should be chosen in the smallest size that still provides a secure fit.

Your gynecologist or specialized physical therapist will determine the right size for you. In the office, you may be fitted with a so-called trial pessary and asked to test whether it fits properly. To do this, press, cough, and move around. You should not be able to feel the pessary. Also make sure that you can urinate and have a bowel movement without any problems while wearing the pessary.

After a few weeks or months, your gynecologist or specialized physical therapist will check whether you need to switch to a different pessary size.

12. Instructions for patients

Please note before use:

- The product may only be used by one patient. It can be used multiple times by one patient.
- Before each use, check the pessary for integrity and cleanliness. If there are any changes to the surface, replace the pessary with a new one.
- We recommend removing the pessary in the evening, cleaning it, and reinserting it in the morning.
- Do not use in cases of bladder, urinary tract, or vaginal infections.
- Contact your gynecologist if you experience any of the following symptoms:
 - ◊ Problems or pain when urinating.
 - ◊ Increase in vaginal discharge or bleeding.
 - ◊ Changes in the color and consistency of vaginal discharge.
 - ◊ Sudden unpleasant odor of vaginal discharge.
 - ◊ Itching or burning in the vagina.
 - ◊ Pessary falls out of the vagina.
- We recommend that all pessary users check the pH value of their vagina regularly using test strips to protect themselves from infections. The pH value of the vagina should always be in the acidic range between 3.8 and 4.5. Regular self-checking can reduce the risk of infection.
- Be sure to attend your gynecological checkups conscientiously.

Inserting the pessary:

- Step 1:** Wash your hands thoroughly before using the pessary.
- Step 2:** Find a relaxed position that suits you best. For example, you can place one foot on a stool or sit on the edge of the toilet seat.
- Step 3:** Moisten the vaginal opening with the enclosed Ladysoft lubricant.
- Step 4:** The pessary is compressed. Insert the pessary along the cervix into the posterior vaginal vault. Then simply push the pessary forward and upward. The pessary is positioned correctly if it does not cause any pressure.

Step 5: Wash your hands afterwards.

Removing the pessary:

Step 1: Wash your hands.

Step 2: Moisten the vaginal area with Ladysoft lubricant.

Step 3: To remove the pessary, get into a comfortable position.

Step 4: Grasp the shell with your index finger and pull the pessary out of the vagina. Do not use any other aids to remove the pessary.

Step 5: Clean the pessary thoroughly with lukewarm water and mild soap or wash lotion.

Step 6: Place the pessary on a towel and allow it to dry.

Step 7: Wash your hands again.

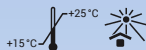
13. Information for healthcare professionals

- The product may only be used by one patient. It can be used multiple times by one patient. The application cycles of a pessary must be documented by the doctor.
- Choice of type and size: It is the responsibility of the gynecologist or specialized physical therapist to determine the type of pessary and the correct size for the patient so that the pessary can exert sufficient pressure on the vaginal walls and alleviate the patient's symptoms.
- The compatibility and correctness of the pessary position must be ensured by ultrasound and/or questioning the patient.
- If a patient cannot urinate with the pessary in place, switch to the next smaller pessary size.
- So-called fitting sets for healthcare professionals, which can be purchased from medesign I.C. GmbH, are used for pessary fitting. These sets contain sterilizable fitting pessaries.
- Complete uterine prolapse often leads to ulceration and erosion when the external cervix or vaginal wall becomes irritated. Any existing irritation

must be treated before starting pessary therapy. A uterine cytology test and/or biopsy are clinically indicated.

- Before inserting the pessary, the prolapsed organs must be pushed back into their anatomically correct position. Please explain to your patient how she can do this herself. Also instruct your patient on how to insert, remove, and clean the pessary herself. Ideally, the patient should insert the pessary in the morning and remove it in the evening.

14. Storage

- Keep out of reach of children.
- Store at room temperature (15-25 °C) and protect from direct sunlight.

15. Shelf life

Best before date of the product as indicated on the product label.

16. Waste disposal/environmental protection

- medesign pessaries are made of medical-grade silicone and can be disposed of in household waste (residual waste bin).
- Silicone is not biodegradable. It must not be disposed of in the toilet.

17. Complementary products

REF	MED1002413	Cleaning solution pessaries 250 ml
REF	MED1002414	Cleaning solution pessaries 500 ml

18. Mandatory text in accordance with the requirements of Regulation (EU) 2017/745 MDR

If you have any questions or problems relating to a pessary, or if you experience serious complications that can be attributed to the pessary, please contact the manufacturer, medesign I. C. GmbH, at info@medesign.de or by phone at +49 (0) 8027 907380. Please notify us immediately if a serious incident has occurred in connection with the product that has, could have, or might have had one of the following consequences, either directly or indirectly:

- the death of the patient, user, or another person,
- the temporary or permanent serious deterioration in the health of a patient, user, or other persons,
- a serious threat to public health.

In the event of a serious incident, you can also contact the competent authority in your country.

19. Explanation of symbols

	manufacturer		date of manufacture
	Attention		batch number
	for reuse on a single patient		Use by
	Follow the instructions for use		latex-free
	item number (MED)		non-sterile
	Protect from sunlight		unique product identification
	temperature limits		CE-mark
	Single Registration Number		medical device

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medesign I.C. GmbH · Dietramszeller Str. 6 · D-83623 Dietramszell