Vaginalgel

Applikator

Vaginal gel applicator

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CE CE-Zeichen

Introduction

Please read the Instructions for use carefully before you start using this medical device, as it contains important information. Always use this medical device exactly as described in these Instructions for Use or exactly as instructed by your physician or healthcare professional.

Follow the safety instructions.


Quick guide for patients

Application instructions


The product may be used by only one patient. It can be used several times by one patient.

1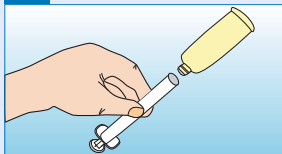


Remove the cap of the ointment tube and put the applicator on/to the tube.

2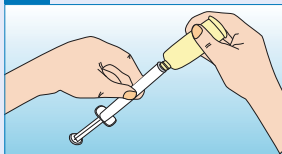


Carefully and slowly squeeze the contents of the tube into the applicator until it is completely filled. Remove the tube from the applicator and close the tube again.

3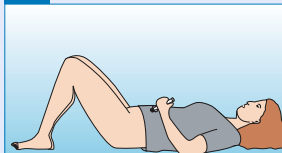


Lie comfortably on your back with your legs bent and spread apart. While doing so, take the filled applicator in one hand.

4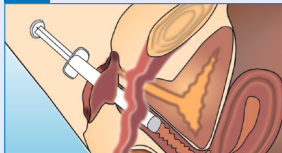

Insert the applicator as deep as you can into your vagina (comparable to a tampon).

5

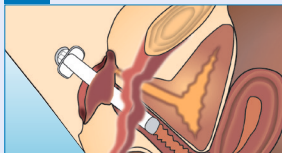Squeeze the applicator empty to deliver the contents from the cylinder into the vagina. Remove the applicator and clean it according to the cleaning instructions.

Purpose

For inserting medicines in gel/cream/ointment form into the posterior vaginal area (Ectocervix).

Safety instructions

- The product may only be used by one patient. It can be used several times by one patient.
- Clean the applicator after each use according to the cleaning instructions.
- Do not use the medical device if it is damaged.

Indications

The vaginal gel applicator facilitates the introduction of vaginal gel, ointments or creams.

Contraindications

Do not use the vaginal gel applicator during pregnancy! In pregnant women, the area around the cervix is very sensitive, so that there is a risk of injury in the event of clumsy handling. Consult your attending physician.

Side effects

There are no known side effects.

Application instructions

The product may be used by only one patient. It can be used several times by one patient.

Detailed application description

1. Wash your hands and dry them carefully.
2. Fill the applicator with the vaginal gel. To do this, open the cap of the tube and put the applicator on/to the tube.
3. Gently and slowly squeeze the contents of the tube into the applicator until it is completely filled.
4. Remove the tube from the applicator and close the tube again.
5. Lie comfortably and relaxed on your back with your legs bent and spread apart. Hold the filled applicator in one hand.
6. Insert the applicator carefully and slowly as deep as you can into your vagina (comparable to a tampon).
7. Squeeze the applicator empty to release the contents from the cylinder into the vagina.
8. Remove the applicator carefully and slowly.
9. Afterwards clean the applicator according to the cleaning instructions and wash your hands.

Cleaning recommendations

After use, clean the applicator with lukewarm (not boiling) water and a mild soap. To facilitate cleaning, disassemble the applicator into its individual parts. To do this, pull the plunger from the cylinder.

Storage

Keep out of reach of children.

Store at room temperature (15-25 °C) and protect from direct sunlight.

Shelf life

Expiration date of the product according to the product labeling.

Waste disposal/environmental protection

After completing treatment, dispose of the applicator in the household waste. Observe your local and national laws, guidelines, standards and specifications for disposal.

Product range

Article number	Item name
REF MED1001731	Tablet applicator vaginal

Required text according to the requirement of Regulation (EU) 2017/745 MDR

In case of questions and problems in connection with the vaginal gel applicator as well as in case of serious complications which can be traced back to the vaginal gel applicator, please contact the manufacturer medesign I. C. GmbH, info@medesign.de or Tel.: +49 (0) 8027 907380. Please report to us immediately if a serious incident has occurred in connection with the product which directly or indirectly had, could have had or could have had one of the following consequences:

- The death of the patient, user or other person,
- The temporary or permanent serious deterioration of the health status of a patient, user or other person,
- A serious public health hazard.

In case of a serious incident, you can also contact the competent authority of your country.

Symbols

REF Item number

LOT Batch number

 Manufacturer

 Observe operating instructions

 Date of manufacture

SN Serial number

 One patient - multiple application

CE CE mark

 Attention

MD Medical device

UDI Unique product identification

 Protect from sunlight

 Temperature limits

SRN Single Registration Number

 Usable until

SRN DE-MF-000020677

UDI

UDI-DI (01) 04250201200044

(17) 301030

LOT

(10) 704273

REF MED1000653

LOT

704273

2030-10

4 250201 200044

Hersteller / Manufacturer:
medesign I.C. GmbH

LOT 704282

SP00646.008

PRN21035