med-erect

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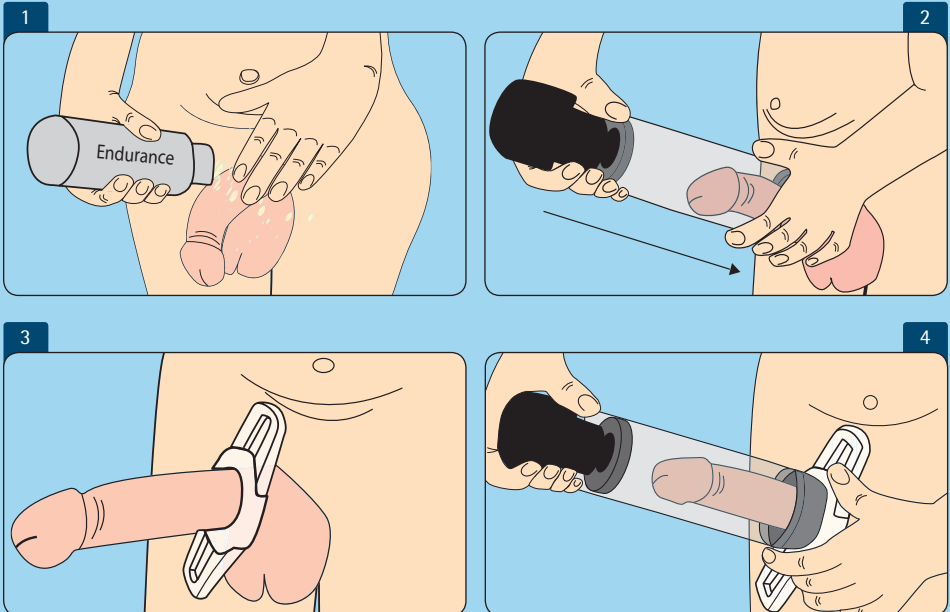
med-erect – vacuum constriction device

This easy to handle device supports your masculinity and allows for successful intercourse. This manual contains all information necessary to achieve an erection with the med-erect vacuum constriction device and should be read carefully. This device is well-proven and allows easy and safe handling. As a start try using the vacuum constriction device on your own until you reach an erection without technical difficulties.

**Impotence and erectile dysfunction are psychological stress for any man.
They can have various causes including:**

- Diabetes
- Cardiovascular diseases
- Troubles with the prostate
- Hormonal disorders
- Multiple sclerosis
- Injuries that affect the spinal cord
- Psychological stress
- Certain prescription medications
- Age
- Alcoholism
- Drug abuse

In any case the causes of erectile dysfunction or impotence should be determined by an urologist or physician. The cause might be identified and even treated.

To be able to reach a satisfying erection without medically treating erectile dysfunction, medesigns med-erect vacuum constriction device is a well-proven medical aid.

The med-erect vacuum constriction device is placed over the penis. The vacuum created in the cylinder of the pump draws blood into the shaft of the penis and causes it to swell and become erect.

Once the penis is erect the constricting band is slid down onto the lower end of the penis to maintain the erection. The constricting band stops the blood from flowing back and thereby makes normal satisfying intercourse possible.

This set includes four different constricting bands made out of soft, highly elastic silicone. When using med-erect for the first few times a middle-sized constricting band should be chosen. The constricting band should not be too tight as this can be uncomfortable and interfere with the feeling of pleasure. If necessary, two constricting bands can be used. They may have the same or different sizes. It is important to choose the right sized constricting band. If it is too large it may allow blood to flow back from the penis which stops the erection. If the band is too small the erection can be rather painful.

Important:

**Do not use the constricting band for more than
30 minutes – this can lead to injury of the blood vessels!
Never sleep when wearing the constricting band!**

To remove the constricting band pull both straps outwards, or simply roll off the band. Pubic hair might get caught in the constricting band and cause pain when removing it. Hair removal or trimming could be helpful in this case.

The constricting band can be reused after cleaning it with liquid soap. However constricting bands may wear out and tear. Therefore it is advisable to always have another fitting band at hand.

Caution:

The constricting band can tear if stretched too much!

Differences between a „natural erection“ and one produced by the med-erect vacuum constriction device:

- The vacuum stretches the entire tissue of his penis whereby the penis can be enlarged.
- is possible that the penis may feel cooler than normal, as the constricting band reduces blood flow.
- The pressure may make blood vessels more visible and cause the penis to turn a darker color.
- The erection of the penis begins where the constricting band is placed which means that the penis does not erect despite the erection.
- The constricting band traps the ejaculate or semen and it will usually dribble out once the constricting band is removed. This does not interfere with the pleasure of a climax or orgasm.
- In some cases the constricting band can cause bruising. This generally will resolve within 24 hours. Should the bruises not disappear or cause pain, do not use the device again and see your doctor.

The med-erect set includes the following:

- Cylinder with a pump and air relief valve
- Reducing ring at the open end of the cylinder
- 4 different sized constricting bands
- 30 ml endurance lubricant
- femsoft intimate lotion
- a Terry towel

The med-erect vacuum constriction device comes assembled and can be used immediately. Should you not be sure which constricting band to use, start with size 3 for the beginning. Apply the endurance lubricant to the bottom ring to allow better sealing when creating the vacuum and for increased lubrication during intercourse.

At the lower end of the constriction device you will find a flexible ring which can be removed to help a large penis fit into the pump easier.

Experience shows that using a vacuum constriction device strains the penis tissue more than usual. The femsoft intimate lotion nourishes the genital area and prevents irritation.

How to create an erection using the med-erect vacuum constriction device.

Tutorial

Practicing the use of the med-erect vacuum constriction device and the constricting bands lets you

achieve an erection sufficient for satisfying intercourse. Before you start using the med-erect vacuum constriction device for intercourse you have to practice how to produce an erection. With a little patience and practice you will achieve great results within a short amount of time. A few tutorials will help you getting to know the device and being able to use it correctly. Furthermore tutorials are necessary to understand what can be expected from the device and to handle the produced erection safely and in a more satisfying way. The specific goal of the exercises is to get the spongy body of penis used to being filled with blood again after a time of not being sexually active. Repeated tutorials allow a slow but steady increase of the amount of blood in the spongy body whereby the achieved erection is increased. At the same time you will learn to extend the duration of the erection.

For the first tutorials you will not need a constricting band. Simply practice producing an erection. After three or four tutorials the spongy body will be used to the vacuum and the larger amount of blood. This is when you can start practice placing the constricting bands at the base of the penis.

Using the med-erect vacuum constriction device

Coat the base of the penis and the adjacent pubic hair thoroughly with the enclosed water soluble endurance lubricant.

You can stand, sit or lie down when using the med-erect vacuum constriction device. Most users find sitting on a chair's edge the most comfortable position.

Take the right sized constricting band and place it over the bottom, open end of the cylinder. Using the rarely used size 1 requires practice while the other sizes can be put into place easily.

Now slowly place the med-erect vacuum constriction device over the penis and firmly press it against the area at the base of the penis coated with lubricant. Then slowly push the pump downwards to create a vacuum. Increase this vacuum by repeated, steady pumping. Allow time for the blood to flow to the spongy body. It may take 4 to 6 minutes until a full erection is achieved.

Caution:

Do not try to further enlarge the penis. This may lead to injury!
med-erect helps producing an erection,
not enlarging the penis!

It may be impossible to create a vacuum in the cylinder if the bottom part does not seal properly at the base of the penis. A reason for this may be excessive pubic hair. Shave the area at the base of the penis and try again by firmly pressing the pump against the body.

If you have achieved a normal erection slide the constricting band from the cylinder onto the penis. Press the air relief valve on the pump and remove the cylinder from the penis. The penis should now stay erect until the constricting band is removed.

If the erection does not last, the constricting band might be too large or air got into the cylinder before the band was slipped on. Choose a smaller constricting band and start creating a vacuum and thereby an erection again.

Should the constricting band cause pain, remove it immediately! Choose a larger constricting band for the next try.

It is normal for the constricting band to cause a little pressure. Constricting bands do not influence the orgasm but may trap the ejaculate. Semen will then only leak from the urethra as soon as the constricting band is removed.

Medesign's constricting bands are made out of a specially developed soft and very elastic silicone. They allow for bruising to be prevented and an ideal erection to be held.

Constricting bands are consumables and may tear or get lost. It is advisable to always have some extra bands in the right size at hand.

Side effects may be

- Bruising on the penis shaft or glans
- Tissue damage
caused by too rapid development of the vacuum

Important notes:

- The med-erect vacuum constriction device can be used in a standing, sitting (chair's edge) or lying down position.
- Take your time when producing the erection. It may sometimes take a couple of minutes.
- Should the penis not erect enough, air might have gotten into the cylinder before the band was slipped on.
- Should the penis not stay erect for long enough, try using a smaller or an additional constricting band.
- Your orgasm will mostly not be accompanied by a visible ejaculation because the constricting band is blocking the urethra. This does not interfere with the climax (orgasm). Semen will only leak after the constricting band is removed.
- Shortly after the orgasm the penis will flag within a few minutes. Make sure to remove the constricting band shortly after the orgasm.
- For hygienic reasons the vacuum constriction device should only be used by one person for their own erection.

The constricting band may not be used for more than 30 minutes!

It has to be removed after this period of time!

Using it longer causes painful and possibly lasting damage!

Do not fall asleep while the constricting band is on the penis!

Cleaning the med-erect vacuum constriction device

The cylinder, distance ring and constricting bands can be cleaned with lukewarm water and liquid soap. The pump should be wiped with a damp towel. Do not place the pump in water or any other liquid.

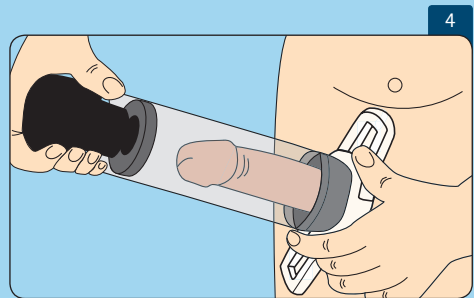
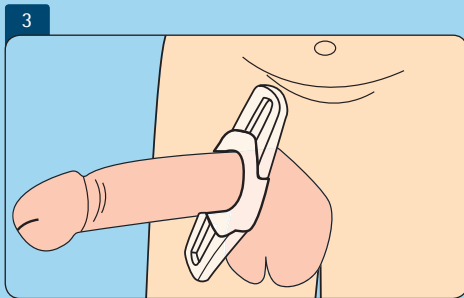
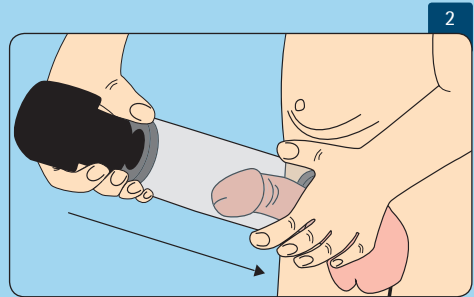
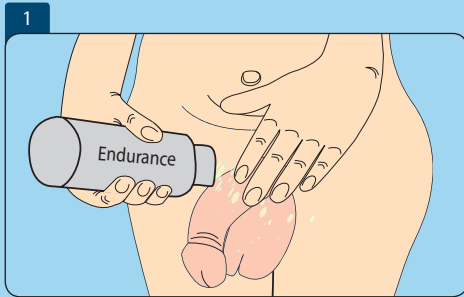
For hygienic reasons the vacuum constriction device and constricting bands are non-returnable items. Only items in the original packaging may be returned.

Disposal of the med erect vacuum pump

All parts can be disposed of in the household waste.

Warranty

There is a legal warranty of 2 years on all parts, except wear parts such as erectionrings. Warranty void if improper use like dropping or exposing the pump head to water.

medesign I.C. GmbH · Dietramszellerstr. 6 · D-83623 Dietramszell
Phone +49(0)8027-907 38-0 · Fax +49(0)8027-907 38-27
info@medesign.de · www.medesign.de