Ayre Spatel

Pap Plus

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Plastik Zervix Abstrichbürsten

Plastic Cervical Scrapers

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Directions for Use (English)

Device description

The Ayre Plastic Spatula gently rubs the mucosa, ensuring a rich sample from ectocervix. The polypropylene plastic is inert and will release collected cells easily onto the slide for pap smear or in the vial for liquid cytology. The Ayre Plastic Spatula is a **single use, non-sterile** device. Discard after use.

Caution

For use by trained medical staff only.

Indications for use

The Ayre Plastic Spatula is used to collect cell samples from the ectocervix for pap smear or liquid cytology. (The endocervical sample is usually collected with a Cytobrush® Plus / medex Cytobrush.)

Contraindications

Do not use on patients suspected of infection, who demonstrate profuse bleeding or are suspected of carrying sexual transmitted diseases.

Sampling instructions

1. Gently fixate the spatula tip in the center of the portio vaginalis.
2. Rotate the spatula slowly one (1) full turn.
3. Withdraw the Ayre Plastic Spatula.

Preparation of sample for pap smear cytology

1. Smear the spatula evenly over a clean glass slide leaving room for the endocervical sample.
2. Fix immediately with cytology spray fixative or 95 % ethanol.

Preparation of liquid cytology

Consult your local laboratory for detailed instructions.

Explanation of symbols

Article number

Use by

Batch code

Attention: See instructions for use

Do not reuse

Product conforms to the Medical Device Directive 93/42/EEC

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medesign I.C. GmbH