# Keulen-Pessar

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CE 0483

Made in Germany

**M E D E S T G N**  
Dietramszell · Dubai · München

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|   |                |    |
|---|----------------|----|
|    | Deutsch .....  | 4  |
|    | English .....  | 11 |
|    | Français ..... | 19 |
|    | Italiano ..... | 27 |
|    | Español .....  | 35 |
|    | Türkçe .....   | 43 |
|    | Polski .....   | 51 |
|    | Русский .....  | 59 |
|  | 中文 .....       | 67 |
|  | عربی .....     | 82 |

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# Club Pessary

Instructions for use

## When is therapy with a Club Pessary useful?

Club Pessaries are used for treating severe descensus or prolapse of the vagina and the uterus e.g. when other pessaries succumb the large pressure and when the use of a Cube Pessary increases involuntary leakage of urine (incontinence). Descents like this can be caused by e.g. hereditary weakness of the connective tissue, postmenopausal hormone deficiency or pregnancy and giving birth.

## How does the Club Pessary work?

medesign's Club Pessary repositions the uterus and brings it into the anatomically correct position – counteracting a prolapse or descensus of the vagina or the uterus (total prolapse). The thickened stem of the pessary is placed behind the symphysis – stabilising the pessary and preventing it from slipping out.

## Which Club Pessary is the right one for me?

medesign's Club Pessaries are offered in various sizes. Selecting the correct size is a prerequisite for ideal treatment and freedom from symptoms. Your gynaecologist will determine the right size for you. Select a pessary, which securely stays in the vaginal vault even when coughing and squeezing. Your gynaecologist will assess the fit of your pessary after a few weeks or months to see whether you can switch to a different size.

medesign pessaries are made from flexible, tissue tolerant silicone, which is especially developed for medical use. If you should, despite this, notice a reaction, stop the use of the pessary and discuss the situation with your physician. The Club Pessary is rather flexible and can thus be inserted without difficulties. The pessary may be used by one patient only.

## How do I insert the Club Pessary?

Pessaries can easily be inserted on your own. Check the Club Pessary for intactness, cleanliness, discolouration, damage and wear marks. The pessary has to be replaced by a new one should damage or changes on the surface be visible.

You should insert the Club Pessary into the vagina in the mornings and remove it in the evenings in order for the sensitive vaginal tissue to be able to recover overnight. In certain cases the pessary may remain inserted during the night or be replaced by a smaller pessary at night. Instead of changing the Club Pessary themselves, many older patients leave this to their physician. Removal and reinsertion is necessary every, or every other week. Please follow your doctor's instructions for this.

Wash your hands prior to inserting the pessary. To ease insertion it is recommended that a lubricant, such as Ladysoft, be used. If needed, the Club Pessary may be coated with a cream or gel containing oestrogen.

To begin with, all prolapsed organs have to be pushed back into their correct position (please have your physician exactly explain all necessary movements for this). Now insert the pessary into the vagina with the handle facing downward. It may be helpful to place one leg e.g. on the edge of your bed. Some women find it easier to insert the Club Pessary in a crouching or lying position. You could also lean against a wall with your legs slightly spread and then insert the pessary. Try out which method is best for you. Use your right hand to gently push the cup - coated with Ladysoft – towards the back and bottom of your vagina. Pay attention to the sensitive bulge of the urethra. Once the ring has slipped past the urethra, turn the cup horizontally and push it as far up as possible. Usually the tip of the stem is still visible.

Push the Club Pessary as far into the vagina as possible or into the position shown to you by your physician. The stem of the Club Pessary should always be placed in a different position on the symphysis in order to avoid pressure marks.

## How do I remove the Club Pessary?

You should also wash your hands prior to removal. Thoroughly lubricate your vaginal area with a large amount of Ladysoft Lubricant. Then pull the stem of the Club Pessary up until it stops at the symphysis. An obstacle in form of a bulge as thick as a finger may form from the loose, overstretched tissue. This bulge must slowly be pushed past the edge of the pessary with your free hand. Following this you will easily be able to remove the Club Pessary. Do not use any assistive equipment for removing the Club Pessary.

## If the Club Pessary slips out when having bowel movement?

It is possible that the pessary slips out partially or totally during defecation. Clean the pessary (as described below) and insert it again as explained above. You can avoid the movement of the pessary by locating the stem of the pessary and holding it in place inside your vagina while pushing.

## Pessary Care

Clean the pessary with lukewarm water and soap or a mild washing lotion such as Femsoft after removal. Subsequently rinse and leave to dry. Disinfection is usually not necessary. Wash your hands again after cleaning the pessary.

For added support for the cleaning and care of your intimate area, use e.g. Femsoft Intimate Wash – a specially developed washing lotion with whey and stinging nettle as a basis and a low pH-value (3.8 – 4). Subsequently wash your hands again.

- Do not use chemicals for cleaning.**
- Do not use household or dish detergents.**
- It is not necessary to boil the pessary.**
- Do not put it in the microwave.**

## Disposal

medesign pessaries are made from certified medical silicone and can be disposed of in household waste. The silicone is not biodegradable and should therefore not be disposed of in the toilet.

**Do not use if you are oversensitive to silicone.  
So far no allergic reactions known.**

**Infections of the vagina and the cervix are contraindications for pessary treatment.**

## Please note

- Read and follow instructions for use.
- Use according to instructions from healthcare professionals.
- Uninterrupted period of application without medical supervision: 29 days maximum.
- Maximum period of application with medical supervision: 2 – 3 cycles of 29 days maximum.
- Repeated use with the same patient possible.
- Tolerance and correct placement of the pessary have to be secured through ultrasound and / or consultation.
- The product has to be checked for intactness and cleanliness of the surface prior to every application. The pessary has to be replaced by a new one should changes on the surface appear.
- The pessary has to be cleaned after every application – using lukewarm water and soap and subsequently rinsing the pessary with clear water.
- The periods of application have to be documented by a physician.
- Do not use during menstruation and never use the pessary together with a tampon or without medical consultation.
- Do not use during pregnancy or if you think you might be pregnant.
- Do not use if you have a bladder or urinary tract infection or inflammation of the vagina.
- medesign pessaries are latex-free.

## Additional products

Ladysoft Lubricant: eases insertion of the pessary

Femsoft Intimate Wash: gentle cleansing of the intimate area

Femsoft Intimate Cream: nourishes and protects the intimate area

**Club Pessary****Size**

|         |            |
|---------|------------|
| Ø 50 mm | MED1000118 |
| Ø 55 mm | MED1000119 |
| Ø 60 mm | MED1000120 |
| Ø 65 mm | MED1000121 |
| Ø 70 mm | MED1000122 |
| Ø 75 mm | MED1000123 |
| Ø 80 mm | MED1000124 |
| Ø 85 mm | MED1000125 |
| Ø 90 mm | MED1000126 |

**Content:**

1 pessary with instruction manual

1 Ladysoft Lubricant with instruction manual

If you have any questions regarding application or would like to share your experiences, please contact us.

Usable until

Item number (MED...)

LOT

Attention

Date of construction

Read and follow the instructions for use

Producer

CE-mark with identification number  
of the notified body

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**medesign I.C. GmbH**
**medesign I.C. GmbH · Dietramszeller Str. 6 · D-83623 Dietramszell**