Finger Feeder

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Finger Feeder

For newborns

Nutritional attachment for single use syringes

Instructions for use

Breast milk is the very best for your baby. If possible every newborn should be breastfed from the very beginning. In certain circumstances, for example when feeding premature babies or newborns with a cleft lip or palate, breastfeeding can be a problem.

With medesign's Finger Feeder it is possible to feed your baby breast milk or supplements in small doses.

Qualified medical personnel should teach and supervise parents using the Finger Feeder. Read instructions carefully and ask qualified medical personnel for advice.

Instructions for use

Pay attention to proper hygiene. Wash hands thoroughly with soap and water before feeding the baby. Fingernails have to be trimmed and filed properly. The Finger Feeder is placed on a disposable syringe filled with breast milk or supplements directly prior to feeding. Make sure the Finger Feeder is clean and the passage is clear.

The baby should be in a slightly upright position, supported by a pillow on the feeding person's lap. Follow the instructions given by the physician or midwife. Gently insert your small finger into the infant's mouth with the underside of the finger facing upwards. While the fingernail rests on the baby's tongue carefully touch the palate with your fingertip in order to trigger the sucking reflex.

The baby should start sucking and swallowing as soon as the fingertip touches the point between the hard and the soft palate. As soon as you can feel the baby's sucking and swallowing movements carefully place the syringe with the Finger Feeder next to your finger – inserted half a centimetre into the corner of the baby's mouth. Now feed the baby breast milk or a supplement drop by drop.

You can easily control the baby's rhythmic sucking and swallowing with your finger. Note the amount fed in order to discuss feeding with your midwife or physician.

Cleaning and sterilising

Clean and sterilise medesign's Finger Feeder prior to first use and always after every use. Let qualified medical personnel advise you. Thoroughly rinse the syringe and Finger Feeder with hot water after every use. The Finger Feeder can be sterilised by boiling it for 3 – 5 minutes, autoclaving it (3 minutes at 134 °C or 20 minutes at 125 °C). When sterilising the Finger Feeder at home it is best to use a sieve over boiling water. Test the Finger Feeder's functionality after every use. It is to be disposed if the formation of drops is not possible.

It is important that:

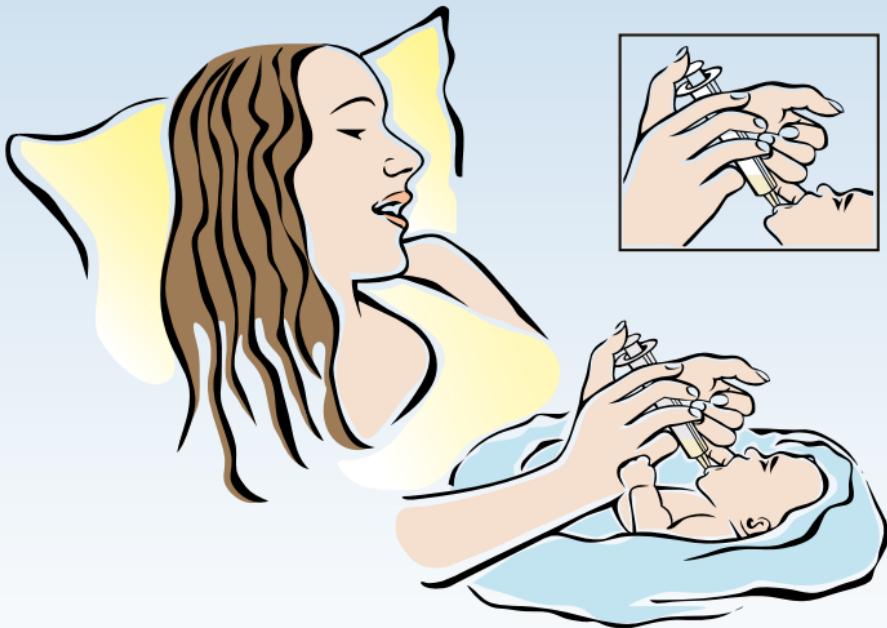
- you are instructed by qualified medical personnel on how to correctly use the Finger Feeder.
- you thoroughly wash your hands and clean your fingernails.
- your fingernails are trimmed.
- you only use cleaned Finger Feeders.
- one or more syringes are filled with breast milk or a supplement directly before feeding the baby.
- the baby is supported in a slightly upright position.
- both of the child's cheeks are stimulated towards the mouth and lips of the baby. This stimulation will ensure that the baby opens its mouth in order to be fed.
- the palate is carefully stimulated with the fingertip.
- you only start feeding the child after feeling sucking and swallowing movements for about 2 minutes.
- you pay attention to the correct movement of the baby's tongue. He or she should only be fed if the tongue moves the way it would during breastfeeding.
- the baby does not suck the Finger Feeder. Prevent wrong sucking by immediately removing the Finger Feeder.

Material

Medical silicone

Disposal

According to local regulations. Normally the Finger Feeder and syringes can be disposed as household waste.

Article	Article No.	EAN
Finger Feeder Content: 3 Finger Feeder, 1 Syringe 10 ml, 1 Syringe 20 ml, 1 Instructions for use	MED1000920	4250201205315
Finger Feeder (Hospital set) Content: 20 Finger Feeder, 1 Instructions for use	MED1000998	4250201205322
Finger Feeder (Hospital set XL) Content: 100 Finger Feeder, 1 Instructions for use	MED1001976	4250201211613

	Usable until		Item number (MED...)
	LOT		Attention
	Date of construction		Read and follow the instructions for use
	Producer		Single patient – multiple use
	Conformity with EU standards		

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medesign I.C. GmbH
medesign I.C. GmbH · Dietramszeller Str. 6 · D-83623 Dietramszell