# Vaginaldehner

## Vaginal Dilator

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## Vaginal dilator:

Vaginal dilators are medical products for conservative therapy in gynaecology, e.g. for treating a narrowing of the entrance to the vagina to avoid surgery. Exercises with the medesign vaginal dilator are usually performed on medical or therapeutic recommendation. The vaginal dilators have proven themselves successful over many years in practice.

## Field of application:

The vaginal dilators are made of flexible, tissue-friendly, soft silicone; or from hard plastic, and are tested and approved for medical use. medesign silicone vaginal dilators are flexible and can be, depending on size, easily and painlessly inserted. Vaginal dilators are used for treating patients which have problems for a variety of reasons.

Such as:

- problematic suture after an episiotomy
- after surgeries involving a reduced size and/or scarring of the vagina
- vaginismus
- during and after radiation therapy
- non-existent vagina (MRKH)
- neovagina
- dyspareunia

The goal is to dilate the vagina. The correct size and shape is decided on together with your physician or therapist. medesign vaginal dilators can have an important effect on a woman's sexuality.

**Application:**

Check the vaginal dilator for integrity, cleanliness, discolouration, damage and wear. In case of visible damage or alterations of the surface, replace the vaginal dilator with a new one.

Wash your hands before using the vaginal dilator. The medesign vaginal dilator is usually prescribed and fitted by a physician. These directions should only be viewed as general information. Application may vary according to your individual situation and should be discussed with your physician or medical attendant. Take your time daily (about 15 to 20 minutes time; maximum 30 minutes) for dilating your vagina. This should be your 'private time' in which you are not disturbed and are in inner peace and relaxation. Dilation of the vagina takes about 3 to 5 months and therefore needs to be planned and prepared. You could perhaps take a relaxing bath first. The warmth softens and loosens the skin and muscles. The better your body is prepared, the more effective the dilation of the vagina will be. We want your efforts to be successful.

**Vaginal dilators round-round / round-conical serve as "place-holders" and can be worn continuously with medical supervision; with a maximum of 2-3 cycles of 29 days each. Ensure hygienic cleaning of the vaginal dilator.**

Should you not be familiar with the anatomy of your intimate area, use a mirror to orientate yourself. Locate the inner and outer labia, the clitoris, the vaginal introitus and the perineum (between the vagina and the anus). Place some lubricant (Ladysoft) on the tip of your index finger and try to insert it into your vagina. The more familiar you are with the anatomy and the sensations of your intimate area, the easier dilation with the medesign vaginal dilator will be.

Find a relaxed seated or reclining position in which you can easily spread your legs. You can use a knee roll or a cushion to support your legs. All your muscles, your entire body even, should be totally relaxed.

Apply the accompanying Ladysoft lubricant to the vaginal introitus and the vaginal dilator. Now guide the tip of the dilator to the vaginal introitus – perhaps using the aid of a mirror – and apply gentle pressure towards the belly button. Stay in that position for a while in order to get used to the feeling of the dilation. Through even insertion and removal of the dilator you will slowly stretch the vaginal tissue. If you feel pain you might be applying too much pressure. You will find the pressure and rhythm that is right for you after a few tries. While you are slowly dilating your vagina your breathing should remain very even. Panting and fast breathing puts your body into a state in which the vaginal muscles are not relaxed.

Pause the exercise until you are totally relaxed again. Allow yourself a break – there is no rush. Once you are familiar with the use of medesign vaginal dilator you can read a book or listen to relaxing music at the same time. If you are able to insert the dilator without feeling any resistance, your vagina has been dilated enough for you to be able to use the next bigger size of vaginal dilator. Try to ‘cling around’ the vaginal dilator with your pelvic floor muscles from time to time. This additional training of the pelvic floor is very beneficial, as the muscle movement activates blood circulation in the entire intimate area. Should you want to end the dilating exercise, remove the vaginal dilator. Allow yourself a few more minutes of total physical relaxation. Clean the vaginal dilator as described in the instruction manual. Regularly check the condition of your vaginal dilator for damage or alterations.

## Side effects / Complications:

When used as intended, there are no known side effects.

You should not be feeling strong pain when inserting or using the vaginal dilator. Should you nevertheless feel intense pain, we strongly advise you to stop the treatment and see a gynaecologist.

Do not use in case of hypersensitivity of silicone, or plastic.

Contraindications to the use of vaginal dilation are infections of the vagina and cervix.

## Cleaning and disinfection:

The vaginal dilator should be used by one patient only.

Thoroughly clean the vaginal dilator with lukewarm water and mild soap or washing lotion after use.

Subsequently place it on a towel to dry or use a hair dryer.

The vaginal dilator can be disinfected with any off-the-shelf liquid disinfectant.

Do not clean with chemical agents. Do not use household detergents or dish-washing liquid. It is not necessary to boil the vaginal dilator. Do not put it in the microwave.

## Please note

- Read and follow instructions for use.
- Use according to instructions from healthcare professionals recommended.
- Uninterrupted period of application without medical supervision:  
29 days maximum (only with round-round / round-conical)
- Maximum period of application with medical supervision:  
2 – 3 cycles of a maximum of 29 days each.
- Repeated use with the same patient possible.
- Tolerance and correct placement of the vaginal dilator can be secured through ultrasound and / or consultation.
- The product has to be checked for intactness and cleanliness of the surface prior to every application. The vaginal dilator has to be replaced by a new one should changes on the surface appear.
- The vaginal dilator has to be cleaned after every application – using lukewarm water and soap and subsequently rinsing the vaginal dilator with clear water.
- Do not use during menstruation and never use together with a tampon or without medical consultation.
- Do not use during pregnancy or if you think you might be pregnant.
- Do not use if you have a bladder or urinary tract infection or inflammation of the vagina.
- medesign vagina dilators are latex-free.

Although it is not known that vaginal dilators increase the risk of TSS (Toxic Shock Syndrome), you should pay attention to it.

## Additional products

- Ladysoft Lubricant: eases insertion of the vaginal dilator
- Femsoft Intimate Cream: nourishes and protects the intimate area

Usable until

Item number (MED...)

LOT

Attention

Date of construction

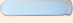
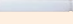
Read and follow the instructions for use

Producer

CE-mark with identification number of the notified body

Areas of application Vaginal dilator		Vaginal stenosis	Surgery and radiation therapy
Vivian V		✓	✓
soft *		✓	✓
Vaginismus			
Neovagina soft *			
round-round *		✓	✓
round-conical *		✓	✓
tapered *		✓	✓
olive shaped		✓	
cylindrical *			✓
cylindrical **			✓
Dobbi Stick **			
Vagitron I **		✓	
Vagitron II **			
Vagitron III *			

\* soft medical silicone    \*\* hard plastic

Cystocele Rectocele	MRKH	Neovagina	Vaginismus	EU standard
			✓	CE
	✓			CE
			✓	CE
✓				CE
✓				CE
				CE
	✓	✓	✓	CE
	✓		✓	CE
✓	✓	✓		CE
		✓		CE
✓				CE
			✓	
			✓	
			✓	

PRN21031

SP00553.003  
24.05.2019

 691650  
2022-07

 Hersteller / Manufacturer:  
medesign I.C. GmbH

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**Copyright ©:** medesign I.C. GmbH