# Ring-Pessar

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CE<sub>0483</sub>

Made in Germany

MEDESIGN

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	Deutsch .....	4
	English .....	11
	Français .....	19
	Italiano .....	27
	Español .....	35
	Türkçe .....	43
	Polski .....	51
	Русский .....	59
	中文 .....	67
	يبرع .....	82

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The background of the page is a solid blue color. On the left side, there is a blurred, light blue silhouette of a woman's back and shoulders. On the right side, there is a large, vibrant yellow sunflower with a dark brown center, partially cut off by the edge of the frame.

# Ring Pessary

 Instructions for use

## When is therapy with a Ring Pessary useful?

Ring Pessaries are used for treating certain types of incontinence (inability to control urination or defecation) as well as problems caused by descensus of the vagina, the uterus, the bladder or the intestine. Descents like this can be caused by e.g. hereditary weakness of the connective tissue, postmenopausal hormone deficiency or pregnancy and giving birth. Treatment with pessaries, combined with pelvic floor exercise and, if necessary, oestrogen enriched creams - can permanently improve conditions and obviate the need for the pessary.

## How does a pessary work?

The medesign pessary repositions the organs and brings them into the anatomically correct position – counteracting a prolapse or descensus of the vagina, the uterus, the bladder or the intestine. Therapy with pessaries is an alternative to surgical intervention for the treatment of descensus uteri and incontinence.

## Which pessary is the right one for me?

Selecting the correct size is a prerequisite for ideal treatment and freedom from symptoms. medesign's Urethra Pessaries are offered in various sizes and models (45 – 100 mm). The pessary, which is smallest in size, yet resting securely should be chosen.

Your gynaecologist will determine the right size for you. The pessary is inserted and you can check its correct position by moving, coughing and squeezing, whereupon the gynaecologist will check the fit of the pessary once again. The pessary, which is smallest in size, yet resting securely should be chosen. You should also make sure that you are able to urinate without any problems.

Your gynaecologist will assess the fit of your pessary after a few weeks or months to see whether you can switch to a different size. The various sizes allow for

individual fitting. Treatment is often carried out with increasing pessary sizes in order to achieve better results. Due to loosening of the tissue, a larger sized pessary often has to be used after 1 - 2 months.

medesign's Urethra Pessaries are made from flexible, tissue tolerant silicone, which is especially developed for medical use. If you should, despite this, notice a reaction, stop the use of the pessary and discuss the situation with your physician. Normal Urethra Pessaries have an integrated metal inlay for stabilization.

The pessaries are flexible and can thus be folded and squeezed, making insertion and removal of the Urethra Pessary especially easy. The pessary may be used by one patient only.

### How do I insert the Ring Pessary?

Pessaries can easily be inserted on your own. Check the Ring Pessary for intactness, cleanliness, discolouration, damage and wear marks. The pessary has to be replaced by a new one should damage or changes on the surface be visible.

You should insert the Ring Pessary into the vagina in the mornings and remove it in the evenings in order for the sensitive vaginal tissue to be able to recover overnight. In certain cases the pessary may remain inserted during the night or be replaced by a smaller Ring Pessary at night. Instead of changing the ring themselves, many older patients leave this to their physician.

Removal and reinsertion is necessary every 4 to 8 weeks. Please follow your doctor's instructions for this.

Wash your hands prior to inserting the pessary. To ease insertion it is recommended that a lubricant, such as Ladysoft, be used. If needed, the Ring Pessary may be coated with a cream or gel containing oestrogen.

Now insert the folded/squeezed Ring Pessary into the vagina. It may be helpful

to place one leg e.g. on the edge of your bed. Some women find it easier to insert the ring in a crouching or lying position. You could also lean against a wall with your legs slightly spread and then insert the pessary. Try out which method is best for you.

Push the pessary into the position shown to you by your physician. Make sure that you place the folded ring in the back vaginal vault and subsequently push it up towards the front. The pessary is placed correctly if you cannot feel any pressure. If the Ring Pessary is not placed correctly it cannot fulfil its supporting function and will slide out under strain.

### How do I remove the Ring Pessary?

You should also wash your hands prior to removal. Thoroughly lubricate your vaginal area with Ladysoft Lubricant. Grab the ring with your index finger and pull the pessary out of your vagina. Do not use any other equipment for removing the Ring Pessary. Subsequently wash your hands again.

## Pessary Care

Clean the pessary with lukewarm water and soap or a mild washing lotion such as Femsoft after removal. Subsequently rinse and leave to dry. Disinfection is usually not necessary. Wash your hands again after cleaning the pessary. For added support for the cleaning and care of you intimate area, use e.g. Femsoft Intimate Wash – a specially developed washing lotion with whey and stinging nettle as a basis and a low pH-value (3.8–4). Subsequently wash your hands again.

Do not use chemicals for cleaning.  
Do not use household or dish detergents.  
It is not necessary to boil the pessary.  
Do not put it in the microwave.

## Disposal

medesign pessaries are made from certified medical silicone and can be disposed of in household waste. The silicone is not biodegradable and should therefore not be disposed of in the toilet.

Ring Pessary (thin) has an integrated metal inlay for stabilization.

This can negatively influence test results of X-rays, CT scans, MRIs, etc. The pessary may heat up during such examinations – inform your physician.

Remove the Ring Pessary (thin) prior to examinations.

Do not use if you are oversensitive to silicone.  
So far no allergic reactions known.

Infections of the vagina and the cervix are  
contraindications for pessary treatment.

### Please note

- Read and follow instructions for use.
- Use according to instructions from healthcare professionals.
- Uninterrupted period of application without medical supervision: 29 days maximum.
- Maximum period of application with medical supervision: 2 - 3 cycles of 29 days maximum.
- Repeated use with the same patient possible.
- Tolerance and correct placement of the pessary have to be secured through ultrasound and/or consultation.
- The product has to be checked for intactness and cleanliness of the surface prior to every application. The pessary has to be replaced by a new one should changes on the surface appear.
- The pessary has to be cleaned after every application – using lukewarm water and soap and subsequently rinsing the pessary with clear water.
- The periods of application have to be documented by a physician.
- Do not use during menstruation and never use the pessary together with a tampon or without medical consultation.
- Do not use during pregnancy or if you think you might be pregnant.
- Do not use if you have a bladder or urinary tract infection or inflammation of the vagina.
- medesign pessaries are latex-free.

## Additional products

Ladysoft Lubricant: eases insertion of the pessary

Femsoft Intimate Wash: gentle cleansing of the intimate area

Femsoft Intimate Cream: nourishes and protects the intimate area

Usable until

Item number (MED...)

LOT

Attention

Date of construction

Read and follow the instructions for use

Producer

Single patient – multiple use

CE-mark with identification number of the notified body

Ring-Pessary thin	Size	Item no.
	∅ 45 mm	MED1001442
	∅ 50 mm	MED1000100
	∅ 55 mm	MED1000101
	∅ 60 mm	MED1000102
	∅ 65 mm	MED1000103
	∅ 70 mm	MED1000104
	∅ 75 mm	MED1000105
	∅ 80 mm	MED1000106
	∅ 85 mm	MED1000107
	∅ 90 mm	MED1000108
	∅ 95 mm	MED1000109
∅ 100 mm	MED1000434	

Ring-Pessary thick	Size	Item no.
	∅ 50 mm	MED1000817
	∅ 55 mm	MED1000818
	∅ 60 mm	MED1000819
	∅ 65 mm	MED1000820
	∅ 70 mm	MED1000821
	∅ 75 mm	MED1000822
	∅ 80 mm	MED1000823
	∅ 85 mm	MED1000824
	∅ 90 mm	MED1000825
	∅ 95 mm	MED1000826
	∅ 100 mm	MED1000827

**Content:**

- 1 pessary with instruction manual
- 1 Ladysoft Lubricant with instruction manual

If you have any questions regarding application or would like to share your experiences, please contact us.

The background is a solid blue color. On the left, there is a faint, light-colored silhouette of a woman's head and shoulders, looking down. On the right, there is a large, vibrant yellow sunflower with a dark brown center, partially cut off by the edge of the frame.

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**medesign I.C. GmbH**