Urethra-Schalen-Pessar

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CE
0483

Made in Germany

M E D E S T I G N
Dietramszell · Dubai · München

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	Deutsch	4
	English	11
	Français	19
	Italiano	27
	Español	35
	Türkçe	43
	Polski	51
	Русский	59
	中文	67
	يَبْرُع	75

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Urethra Bowl Pessary

 Instructions for use

When is therapy with an Urethra Bowl Pessary useful?

Urethra Bowl Pessaries are used for treating certain types of incontinence (inability to control urination or defecation) as well as problems caused by descensus of the vagina, the uterus, the bladder or the intestine. Descents like this can be caused by e.g. hereditary weakness of the connective tissue, postmenopausal hormone deficiency or pregnancy and giving birth. Treatment with pessaries – combined with pelvic floor exercise and, if necessary, oestrogen enriched creams – can permanently improve conditions and obviate the need for the pessary.

How does the Urethra Bowl Pessary work?

The medesign pessary repositions the organs and brings them into the anatomically correct position – counteracting a prolapse or descensus of the vagina, the uterus, the bladder or the intestine. Therapy with pessaries is an alternative to surgical intervention for the treatment of descensus uteri and incontinence. The thickenings on the upper edge of the pessary (= calotte) elevate the bladder neck and thereby prevent involuntary discharge of urine when the abdominal pressure is increased during movements like e.g. coughing. The bowl creates a large contact area on the pelvic floor, demanding for the latter to be at least somehow stable.

Which pessary is the right one for me?

Selecting the correct size is a prerequisite for ideal treatment and freedom from symptoms. medesign's Urethra Bowl Pessaries are offered in various sizes (55 – 85 mm). The pessary, which is smallest in size, yet resting securely should be

chosen. Your gynaecologist will determine the right size for you.

The pessary is inserted and you can check its correct position by moving, coughing and squeezing, whereupon the gynaecologist will check the fit of the pessary once again. You should also make sure that you are able to urinate without any problems.

Your gynaecologist will assess the fit of your pessary after a few weeks or months to see whether you can switch to a different size. The various sizes allow for individual fitting. Treatment is often carried out with increasing pessary sizes in order to achieve better results. Due to loosening of the tissue, a larger sized pessary often has to be used after 1 – 2 months.

medesign's Urethra Bowl Pessaries are made from flexible, tissue tolerant silicone, which is especially developed for medical use. If you should, despite this, notice a reaction, stop the use of the pessary and discuss the situation with your physician. The pessaries are flexible and can thus be folded and squeezed, making insertion and removal of the Urethra Bowl Pessary especially easy. The pessary may be used by one patient only.

How do I insert the Urethra Bowl Pessary?

Pessaries can easily be inserted on your own. Check the Urethra Bowl Pessary for intactness, cleanliness, discolouration, damage and wear marks. The pessary has to be replaced by a new one should damage or changes on the surface be visible.

You should insert the Urethra Bowl Pessary into the vagina in the mornings and remove it in the evenings in order for the sensitive vaginal tissue to be able to recover over night. In certain cases the pessary may remain inserted during the night or be replaced by a smaller Urethra Bowl Pessary at night. Instead of

changing the pessary themselves, many older patients leave this to their physician. Removal and reinsertion is necessary every 4 to 8 weeks. Please follow your doctor's instructions for this.

Wash your hands prior to inserting the pessary. To ease insertion it is recommended that a lubricant, such as Ladysoft, be used. If needed, the calotte and the Urethra Bowl Pessary may be coated with a cream or gel containing oestrogen.

Now insert the Urethra Bowl Pessary into the vagina. It may be helpful to place one leg e.g. on the edge of your bed. Some women find it easier to insert the Urethra Bowl Pessary in a crouching or lying position. You could also lean against a wall with your legs slightly spread and then insert the pessary. Try out which method is best for you.

Push the pessary into the position shown to you by your physician. Make sure that the calotte is placed exactly below the urethra. The pessary is placed correctly if you cannot feel any pressure. If the Urethra Bowl Pessary is not placed correctly it cannot fulfill its supporting function and will slide out under strain.

How do I remove the Urethra Bowl Pessary?

You should also wash your hands prior to removal. Thoroughly lubricate your vaginal area with Ladysoft lubricant. Simply move the calotte aside and grab the Urethra Bowl Pessary with your index finger to pull the pessary out of your vagina. Do not use any other equipment for removing the Urethra Bowl Pessary. Subsequently wash your hands again.

What do I do if the pessary does not stop the leakage of urine?

If the inserted pessary does not stop the leakage of urine, the placement of the calotte has to be checked or you might have to change to a different pessary size or model.

Pessary Care

Clean the Urethra Bowl Pessary with lukewarm water and soap or a mild washing lotion such as Femsoft after removal. Subsequently rinse and leave to dry. Disinfection is usually not necessary. Wash your hands again after cleaning the pessary.

For added support for the cleaning and care of your intimate area, use e.g. Femsoft Intimate Wash – a specially developed washing lotion with whey and stinging nettle as a basis and a low pH-value (3.8 – 4). Subsequently wash your hands again.

- Do not use chemicals for cleaning.
- Do not use household or dish detergents.
- It is not necessary to boil the pessary.
- Do not put it in the microwave.

Disposal

medesign pessaries are made from certified medical silicone and can be disposed of in household waste. The silicone is not biodegradable and should therefore not be disposed of in the toilet.

Infections of the vagina and the cervix are contraindications for pessary treatment.

Please note

- Read and follow instructions for use.
- Use according to instructions from healthcare professionals.
- Uninterrupted period of application without medical supervision: 29 days maximum.
- Maximum period of application with medical supervision: 2 – 3 cycles of 29 days maximum.
- Repeated use with the same patient possible.
- Tolerance and correct placement of the pessary have to be secured through ultrasound and / or consultation.
- The product has to be checked for intactness and cleanliness of the surface prior to every application. The pessary has to be replaced by a new one should changes on the surface appear.
- The pessary has to be cleaned after every application – using lukewarm water and soap and subsequently rinsing the pessary with clear water.
- The periods of application have to be documented by a physician.
- Do not use during menstruation and never use the pessary together with a tampon or without medical consultation.
- Do not use during pregnancy or if you think you might be pregnant.
- Do not use if you have a bladder or urinary tract infection or inflammation of the vagina.
- medesign pessaries are latex-free.

Additional products

Ladysoft Lubricant: eases insertion of the pessary

Femsoft Intimate Wash: gentle cleansing of the intimate area

Femsoft Intimate Cream: nourishes and protects the intimate area

Usable until

LOT

Date of construction

Producer

Item number (MED...)

Attention

Read and follow the instructions for use

CE-mark with identification number
of the notified body

Urethra Bowl Pessary	Size	Item No.
	Ø 55 mm	MED1000071
	Ø 60 mm	MED1000072
	Ø 65 mm	MED1000073
	Ø 70 mm	MED1000074
	Ø 75 mm	MED1000075
	Ø 80 mm	MED1000076
	Ø 85 mm	MED1000077

Content:

- 1 pessary with instruction manual
- 1 Ladysoft Lubricant with instruction manual

If you have any questions regarding application or would like to share your experiences, please contact us.

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medesign I.C. GmbH
medesign I.C. GmbH · Dietramszeller Str. 6 · D-83623 Dietramszell