Analdehner

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 CE-Zeichen

Introduction

Please read the Instructions for use carefully before you start using this medical device because it contains important information.

Always use this medical device exactly as described in these instructions for use or exactly as instructed by your doctor, paediatrician or healthcare professional.

Follow the safety instructions.

Quick guide for patients

Application instructions for patients

The product may only be used by one patient. It can be used several times by one patient.

Application description

1. Before use: Defecation, thoroughly clean the anal region, wash hands.
2. Moisten anal dilator with a little lubricant.
3. Lie on your side and bend both knees or lie in a relaxed supine position.
4. Slowly insert the anal dilator into the anal canal with your hand while gently rotating it. Then pull it out again.
5. Repeat the insertion 5-10 times. It is best to do this regularly in the morning and evening.
6. With regular practice, the anal dilator should be held in the anal canal for three minutes at last – as deep as possible.
7. After each use, clean the anal dilator with lukewarm water and a soap solution or a mild washing lotion. Then rinse with clear water and let dry.
8. Wash your hands again after cleaning the anal dilator.

Purpose

For conservative therapy of anal stenosis.

Rectal disorders are common and affect all age groups. Surgical treatment of anal or rectal fistulas or haemorrhoids provides relief. Sometimes this can narrow the anal canal, causing pain during bowel movements. In children, the opening of the anus may be narrowed, covered with skin or completely absent (anorectal malformation). The anal dilator is used to carefully widen the anal canal and sphincter and improve its elasticity. Medically, the anal dilator is used to overcome a reflex spasm of the sphincter muscle in acute anal fissure, thus assisting its healing. The anal dilator is an instrument made of hard plastic or silicone. It is usually inserted with a medical lubricant (e.g. m-lys).

Safety instructions

- The product may only be used by one patient. It can be used several times by one patient.
- Clean the anal dilator after each use according to the cleaning instructions.
- Do not use the medical device if it is damaged.
- Be sure to consult your physician if you experience pain, itching or bleeding. Go for an annual check-up, because the timely detection of rectal diseases increases the chance of recovery.

Indications

The anal dilators can be used for the following indications:

- For haemorrhoids
- Anal fistula
- Anal fissure
- Anorectal malformations

Contraindications

No contraindications are known.

Side effects

At the beginning of use, insertion of the anal dilator may be associated with slight pain and/or occasional light bleeding.

Application instructions

- The product may only be used by one patient. It can be used more than once by one patient.
- Use as directed by a physician, paediatrician, or health care professional.
- Before each application, check the product for integrity and cleanliness of the surface. In case of changes in the surface, replace the anal dilator with a new one.
- The duration of use must be documented by the physician or paediatrician.
- medesign's anal dilators are latex-free.

Detailed application description

Before using the product for the first time, be sure to follow your doctor's instructions physician's instructions regarding the type, frequency and duration of use of the anal dilator.

Check the anal dilator for integrity, cleanliness, discolouration, damage and wear. If there is visible damage or discolouration of the surface, replace the anal dilator with a new one.

Before use, defecation and thorough cleaning of the anal region should be performed, e.g. with hygienic wet wipes or an intimate washing lotion (e.g. Femsoft).

Before insertion, you should wash your hands. To make insertion easier, the use of a lubricant such as m-lys has proven effective. Apply the medical lubricant to the tip of the anal dilator as needed. Lie on your side with both knees slightly drawn up or in a relaxed supine position. Slowly insert the anal dilator into the anal canal with your hand while gently rotating it. This spreads the gel over the entire anal dilator and over the skin of the lower anal canal, making insertion easier. At first, insertion of the anal dilator may be associated with slight pain; after all, the goal is to widen an anal canal that is too narrow and inelastic.

Therefore, insert the dilator only a short distance at first. In the following days, the anal dilator can then be inserted further and further. Occasional slight bleeding is no cause for concern. Complications are not to be expected if used properly. Regular application is necessary for the success of the treatment. The best time is always in the morning and evening. First, slowly insert the anal dilator several times (5-10 times). Finally, it should be held – as deeply as possible – in the anal canal for three minutes.


As an additional training for the sphincter muscle, you should squeeze it together hard several times against the resistance of the anal dilator. This leads to a strengthening of the sphincter and trained pelvic floor muscles.

Cleaning recommendations

- After use, clean the anal dilator with lukewarm (not boiling) water and a soap solution or mild washing lotion. Then rinse with clean water and allow to dry. Disinfection is usually not necessary. After cleaning the anal dilator, wash your hands again.
- Do not clean with chemical agents. Do not use household cleaners or detergents. It is not necessary to boil the anal dilator. Do not place it in the microwave.

Storage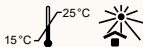

- Keep out of reach of children.
- Store at room temperature (15-25 °C) and protect from direct sunlight.

Shelf life

Expiration date of the product according to the product labelling.

Waste disposal/environmental protection

After completing treatment, dispose of the applicator in the household waste. Comply with your local and national laws, guidelines, standards and regulations for disposal.

Product range

Article number	Item name	Size (mm)
REF MED1001738	Anal dilator hard plastic olive shape	∅ 10 × 6 × L 53
REF MED1001739	Anal dilator hard plastic olive shape	∅ 13 × 7 × L 58
REF MED1001416	Anal dilator hard plastic olive shape	∅ 15 × 8 × L 61
REF MED1001740	Anal dilator hard plastic olive shape	∅ 18 × 10 × L 74
REF MED1001741	Anal dilator hard plastic olive shape	∅ 20 × 12 × L 76
REF MED1001742	Anal dilator hard plastic olive shape	∅ 22 × 14 × L 79
REF MED1001743	Anal dilator hard plastic olive shape	∅ 25 × 16 × L 84
REF MED1000747	Anal dilator silicone conical	Size 1, ∅ 4 × 75 mm
REF MED1000748	Anal dilator silicone conical	Size 2, ∅ 6 × 75 mm
REF MED1007479	Anal dilator silicone conical	Size 3, ∅ 8 × 75 mm
REF MED1000750	Anal dilator silicone conical	Size 4, ∅ 10 × 75 mm
REF MED1000751	Anal dilator silicone conical	Size 5, ∅ 12 × 75 mm
REF MED1000752	Anal dilator silicone conical	Size 6, ∅ 14 × 75 mm
REF MED1000753	Anal dilator silicone conical	Size 7, ∅ 16 × 75 mm
REF MED1000759	Anal dilator silicone conical	Size 8, ∅ 18 × 75 mm
REF MED1001519	Anal dilator silicone conical	Size 9, ∅ 20 × 75 mm
REF MED1001520	Anal dilator silicone conical	Size 10, ∅ 22 × 75 mm
REF MED1001521	Anal dilator silicone conical	Size 11, ∅ 24 × 75 mm
REF MED1001522	Anal dilator silicone conical	Size 12, ∅ 26 × 75 mm
REF MED1001523	Anal dilator silicone conical	Size 13, ∅ 50 × 75 mm

Mandatory text according to the requirement of Regulation (EU) 2017/745 MDR

In case of questions and problems in connection with the anal dilator as well as in case of serious complications which can be traced back to the anal dilator, please contact the manufacturer medesign I. C. GmbH, info@medesign.de or phone: +49 (0) 8027 907380. Please report to us immediately if a serious incident has occurred in connection with the product which directly or indirectly had, could have had or could have one of the following consequences:

- The death of the patient, user or other person,
- The temporary or permanent serious deterioration of the health status of a patient, user or other person,
- A serious public health hazard.

In case of a serious incident, you can also contact the competent authority of your country.

Content:

- 1 anal dilator plastic with instructions for use
1 m-lys lubricant

Symbole

 REF Item number

 LOT Batch number

 Manufacturer

 Observe operating instructions

 Date of manufacture

 Usable until

 One patient - multiple application

 CE mark

 Attention

 MD Medical device

 UDI Unique product identification

 Protect from sunlight

 Temperature limits

 SRN Single Registration Number

SRN DE-MF000020677

 medesign I.C. GmbH · Dietramszeller Str.6 · D-83623 Dietramszell
www.medesign.de · info@medesign.de

PRN21331

SP02652.000
01.06.2023

 Lot700851
2028-06

 Hersteller / Manufacturer:
medesign I.C. GmbH