Rapssamenkissen

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Rape Seed Bag

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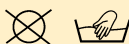
Rape Seed Bag

for gentle massage and thermo-therapy

medesign's Rape Seed Bag is a traditional household remedy used in the same way as a hot water bottle. Therefore it also goes by the name of waterless hot water bottle. When the Rape Seed Bag is heated up, the pits inside the Bag will retain heat much longer than water in a hot water bottle. Due to its design, the Rape Seed Bag will perfectly adapt to the body's shape allowing for perfect thermal transmission.

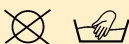
Heating:

In the oven at a maximum temperature of 100 °C. In the microwave, without grilling or hot air function (danger of combustion!). Start with the lowest level and increase gradually until the bag has been heated up to a pleasant temperature. Please check your microwave manual. Where less heat is needed, the bag can also be laid on the radiator.

Cooling:

medesign's Rape Seed Bag may also be used to provide cooling. Simply put the Bag in a freezer or refrigerator.

Furthermore, the medesign Rape Seed Bag can be used in massages during pregnancy or for general hand and foot exercises. This is especially helpful when mobility of the extremities has to be restored through specifically applied massage, or activated in children.

Artikel Nr.	Artikel	Größe / Form
MED1000986	Rapssamenkissen gelb	12 × 16 cm
MED1000987	Rapssamenkissen gelb	22 × 22 cm
MED1000988	Rapssamenkissen gelb	40 × 20 cm
MED1000989	Rapssamenkissen gelb	40 × 30 cm
MED1000990	Rapssamenkissen gelb	40 × 40 cm
MED1000991	Rapssamenkissen gelb	40 × 50 cm
MED1000992	Rapssamenkissen gelb	40 × 60 cm
MED1001091	Rapssamenkissen gelb	40 × 80 cm
MED1000993	Rapssamenkissen blau	'Fisch'
MED1000994	Rapssamenkissen rot	'Herz'
MED1001048	Rapssamenkissen gelb	'Maus'
MED1000995	Rapssamenkissen gelb	'Mond'
MED1000997	Rapssamenkissen gelb	'Nackenhörnchen'
MED1000996	Rapssamenkissen gelb	'Stern'
MED1000984	Rapssamen Nachfüllbeutel	1 kg

Hersteller / Manufacturer:
medesign I.C. GmbH

Lot# 696035
2026-01

SP00645.003
07.01.2021

PRN21100